



So long, scar. Hello, beautiful.

DISTRIBUTED BY:
DENEPO, LLC
6070 Vinyard Lane
Cumming, GA 30041

www.InvisiScar.com
www.info@invisiscar.com



So long, scar. Hello, beautiful.

The InvisiScar™ patch is a soft, durable and reusable medical-grade silicone patch. It is designed as a low-cost means to non-surgically manage and naturally reduce the appearance of hypertrophic and keloid scars resulting from surgery, burns, piercing, cystic acne and cuts. Almost any scar, regardless of age and severity, may be treated with the InvisiScar™ patch. It can be used with most any skin color or type.

After using the InvisiScar™ patch for a period of time—a period which varies depending on the severity and age of the scar—you should notice a reduction in the irregular color, flatness, and appearance of the scar. Although the InvisiScar™ patch will work on any scar regardless of age, the time it takes to reduce the scarring will increase based on the age of the scar.



So long, scar. Hello, beautiful.

HOW TO USE THE INVISISCAR™ PATCH

1. Thoroughly wash both the InvisiScar™ patch and scar area with mild soap and water before each use. Allow to completely dry before proceeding.
2. If needed, cut the InvisiScar™ patch to fit the scar area, allowing at least a 1/4" to 1/2" overlap of the patch around the edges of the scar. (You may use either side of the patch against the scar area.) Secure the patch on the scar with hypoallergenic tape.
3. Start by using the InvisiScar™ patch 1-2 hours per day for the first 2 or 3 days. Then increase the time the patch is applied to the scar area until you can wear it for as many hours each day as possible.

The InvisiScar™ patch may be used for up to 6 months. After that, a new patch should be applied. Depending on the severity and age of the scar, it may take anywhere from a few weeks to several months of using the InvisiScar™ patch to achieve results.



So long, scar. Hello, beautiful.

WARNINGS & PRECAUTIONS

Do not use the InvisiScar™ patch on any open wound.

Thoroughly wash both the InvisiScar™ patch and scar area with mild soap and water before each use. Allow to completely dry before application.

As with any device attached directly to the skin, some type of reaction may occur. If the skin around the scar becomes red or begins to itch, discontinue use of the InvisiScar™ patch for a few days. If any redness and/or itching reoccurs during subsequent use, consult your pharmacist. You may need to replace the hypoallergenic tape supplied in this package with one that better matches your skin type.

The InvisiScar™ patch must always be kept clean and dry to prevent the growth of unwanted bacteria.

www.InvisiScar.com